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Instructions for use and care

Glocken Medizinball - Klang-Medizinball

Congratulations on the purchase of your Glocken medicine ball!

With the Glocken medicine ball you have chosen a high-quality TOGU® brand product 'made in Germany'!

The sound medicine ball is latex-free, contains no heavy metals and is made of 100% recyclable plastic - in proven TOGU quality.

It was specially developed for hearing training for blind people and for coordination and sensory training.

Thanks to its integrated sound hole, the ball provides acoustic feedback during movement. The textured surface ensures safe handling - even with impaired vision.

A little equipment information:

The bell medicine ball is made of high-quality Ruton® and is equipped with a sound opening.

Dimensions: approx. 21 cm diameter

Weight: approx. 800 g

Load capacity: up to approx. 200 kg

Special features:

- Latex-free and hygienic
- With acoustic sound function for ear training
- Ideal for the blind and visually impaired

Temperature resistance:

The bell medicine ball is UV-resistant and can be used at temperatures of up to 45 °C. Please do not use in direct sunlight.

Please do not store in direct sunlight - heat can soften the material.

Also resistant to cold, but should not be stored below -10 °C as the material becomes firmer in the cold.

Care:

All mild, commercially available cleaning agents and disinfectants can be used for cleaning.

Please do not use any solvent-based, acidic or corrosive cleaning agents.

The ball is waterproof.

Storage:

Store the bell medicine ball in a dry, clean place at normal room temperature. Avoid direct heat, frost and sharp objects.